

SENTINEL service checkout

Quick check - when recycling SENTINEL pumps into patient service. No tube set installed.

- 1) before starting, plug pump in AC outlet for at least 10hr to charge battery.
- 2) check that pump is clean and there is no accumulated debris, especially in area of rotor and drop sensor openings (two small holes where drip chamber is placed).
- 3) with pump unplugged, press ON then press RUN to check that pump operates on battery.
 - If there is an IMMEDIATE alarm and Low Bat light is flashing, battery must be replaced – call service.
 - If there is no immediate alarm, press OFF, go to next step.
- 4) plug pump into AC outlet and press ON.
 - If Bat light is on, there is no AC power OR internal fuse is blown – call service.
 - If Bat light is off, go to next step.
- 5) Press and hold UP button until rate display shows 295. Press RUN and wait. Rotor should turn.
 - If pump runs for approximately 15 sec and then alarms, pump is Ok to use.
 - If “FLO FAST” alarm appears on display, do not use pump – call service.
- 6) For a pump that will be used WITHOUT setting a dose – Press ON, then press SET DOSE button.
 - If right display is 0, pump is Ok to use. Press OFF.
 - If right display is NOT 0, press and hold CLEAR until display is 0. Press OFF.